

Freeway Transactive Xtra CTH Troubleshooting Guide

Scenario: Hoist LCD is not functioning

When operating the hand controls, the hoist has power and performs the usual lifting and lowering actions, but the LCD is not functioning

Are there any visible defects to the LCD and its wiring?

YES

Replace the LCD (bottom cover) assembly

Defects include cracked casing, pixel damage burns or damaged wiring

NO

Unresolved?

Is the LCD cable connected to the PCB?


NO

Plug the LCD cable back into the PCB port

Unresolved?

YES

See PCB and LCD Testing Guide - Document No. 990SD-12

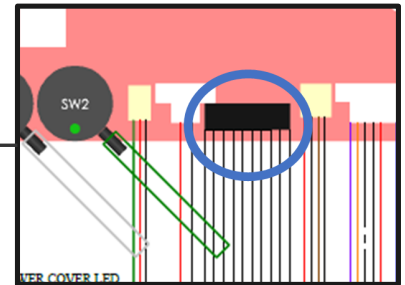

Key:

Question

Action

Component Replacement

Supporting Document

Additional Context and Actions