


Mackworth CPP Portable Lift Troubleshooting Guide

Scenario: Battery Temperature Sensor Fault

When operating the hand controls, the portable lift LED indicates a yellow and blue light and an audible alarm will repeat 5 beeps (0.5s apart) x2 cycles.

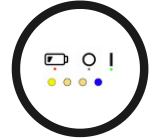

When this scenario occurs, there must be damage to the battery internals or the sensor wires which is all part of the battery assembly.

Replace the batteries as a pair

If the damage has occurred to the sensor wire section found on the toggle switch, then relace the toggle switch assembly

Key:

Question

Action

Component
Replacement

Supporting
Document

Additional Context
and Actions