Troubleshooting Guide Doc No. 990097-10

Mackworth CP Portable Lift Troubleshooting Guide Scenario: Battery Voltage Delta Exceeded

Revision: A

Revision Date: 08/24

When operating the hand controls, the portable lift LED indicates a yellow and blue light and an audible alarm will repeat 4 beeps (0.5s apart) x2 cycles.



When this scenario occurs, it means that either one or both batteries aren't holding charge as expected

Replace the batteries as a pair

Key:

Ouestion

Action

Component Replacement Supporting Document

Additional Context and Actions