

# Mackworth CP Portable Lift Troubleshooting Guide

## Scenario:


### Portable Lift does not raise and lower - No Error

When operating the hand controls, the portable lift doesn't raise and lower, and there is no error indication.

If there is excessive wear between the frictional components (motors and hub), the teeth on the hub may become so worn and there is no contact made between the mating parts, causing the motor to spin during operation but no lifting will occur.

Replace worn components.  
(e.g. Lift Motor, Hub (Spool) and  
Lift Tape

Unacceptable Condition - Do not use

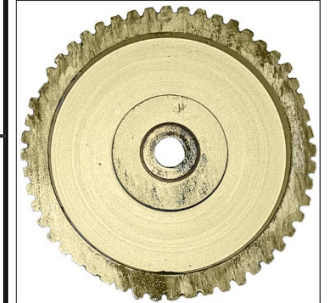

## Key:

Question

Action

Component  
Replacement

Supporting  
Document

Additional Context  
and Actions