Ceiling Lift does not raise and lower - High Current When operating the hand controls, the ceiling lift LCD indicates "High Current" and an audible alarm will continuously beep for 3 seconds. YES-When the SWL (Safe Working Load) is exceeded The SWL can be located on the main label and the ceiling lift will cut power to avoid excessive in the user manual. loading NO -Unresolved?-No. 990SD-01 The ampage can be checked in the program Retest the function by operating the hand node. The following SWL's should be set as the controls SWL 440lb - 20A SWL600lb - 24A -Unresolved?-Common wear is the lift tape fraying within the gearing, or -Excessive wear? excessive wear to the hub Retest the function by operating the hand controls Re-grease the hub and motor shaft and retes the function by operating the hand controls -Unresolved?-If the motor fails to rotate, there has been a gear box failure Retest the function by operating the hand Key: Additional Context and Actions