



Finger Lakes
Porter




Finger Lakes Porter

The Finger lakes Porter comes with all the support and comfort features of our healthcare chairs, but with the added advantage of a Tilt-in-Space recline mechanism with an independent backrest and slide out footrest, perfect for ease of movement. The chair is controlled by an easy-to-use handset and the chair can be smoothly reclined and repositioned by the user or carer.


Back Cushion

The Finger Lakes Porter comes with our interchangeable backs. Multiple back cushions can be purchased to support the needs of different individuals.

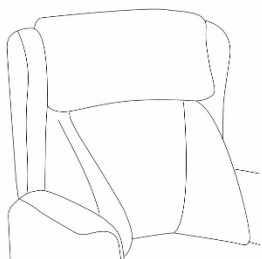

Fiber Lateral

Individual fibre-filled waterfall cushions, each of which is fully adjustable to accommodate maximum postural support.

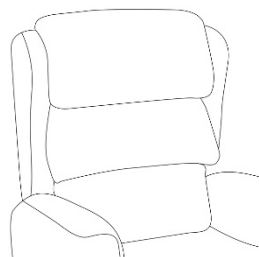

Waterfall


Soft fibre filled back with lateral support, which can be useful for those who have limited trunk control or balance.

Seat Cushion

This seat cushion uses the latest medical-grade silicone gel technology, allowing pressure to be evenly distributed. For added support and softness, a layer of premium memory foam is built in above the gel base. Additionally, the cushion includes a chaise layer.

Coolform

- Coolform Jel
- Chaise Layer
- Memory Foam
- Reflex Foam

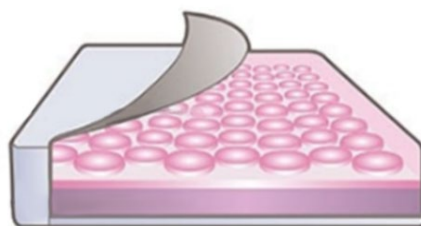

The gel conforms to the shape of the body and equalises to body temperature. The gel cells allow air to flow easily and Coolform is totally odourless and hypoallergenic.

Mechanism

The mechanism includes a tilt-in-space recline, a backrest that moves independently, and a slide-out footrest. This gives users control over how they sit, helping them find the most comfortable and supportive position.



Tilt-in-space

This permits the angle of the hips, knees, and ankles to stay constant during the movement of the entire seating system leaned backwards while maintaining the same seat back angle.



Independent Backrest

The backrest recline can be adjusted separately to achieve a comfortable position. The reclining option can be modified from an upright position to a reclined angle, allowing the caregiver to change the user's back position and expand the hip angle for added comfort.



Slide-out Footplate

The chair features a slide-out footplate for use when transporting an occupant from one location to another

Size Guide

The Finger Lakes Porter has a weight limit of 280lb (20 stone) as standard.

Seat height	Seat width	Seat depth	Back height	Arm height
26"	20"	17"	20.5"	24.5"

Fabric

There is a range of fabrics to choose from in the Valencia, Cadet, and Dartex VP ranges available. All our health fabrics are Waterproof, Anti-MRSA, Anti-bacterial, Anti-fungal, Crib 5, and bleach cleanable. All cushions and cushion covers are removable for cleaning purposes.

Accessories

The Finger Lakes Porter comes as standard with:

- A client handset loop to the right-hand side arm
- Full-length arm caps with a right-hand side handset loop
- A carer loop to the rear right-hand side wing of the chair
- An elasticated flowerpot pocket to the rear of the chair
- Heavy-duty castors
- A push handlebar
- A sliding footplate



